



# ABSORBENT DRESSING WITH SAFETAC<sup>®</sup> TECHNOLOGY

- Atraumatic to the wound and surrounding skin on removal
- Absorbs effectively
- Can be used under compression
- Minimises the risk of maceration



## AREAS OF USE

Mepilex is designed for a wide range of exuding wounds such as leg and foot ulcers, pressure ulcers and traumatic wounds, e.g. skin tears and secondary healing wounds.

## NOTE

In case of clinical signs of infection the use of Mepilex may be continued if proper infection treatment is initiated.

## HOW TO USE MEPILEX



1. Clean the wound area. Remove the release film.



2. Mepilex should overlap the wound bed by at least 2 cm onto the surrounding skin. Apply the adherent side to the wound. Do not stretch.



3. When necessary, secure Mepilex with a bandage or other fixation.

## BENEFITS OF MEPILEX

- Atraumatic to the wound and surrounding skin on removal
- Can remain in place for several days depending on the condition of the wound
- Minimises the risk of maceration
- Promotes patient comfort during wear
- Can be used under compression bandages
- Can be cut to size

### MEPILEX ORDERING INFORMATION

Art. no	Size cm	Pieces per inner	PIP Code	NHS Code
294160	10 x 11	5	339-1109	ELA378
294260	11 x 20	5	339-1091	ELA085
294360	15 x 16	5	339-1083	ELA333
294460	20 x 21	5	339-1034	ELA334
294500	20 x 50	2	342-9537	ELA383



Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, [www.molnlycke.co.uk](http://www.molnlycke.co.uk)  
The Mölnlycke Health Care name and logo, Safetac® and Mepilex® are registered trademarks of Mölnlycke Health Care AB. ©2009 Mölnlycke Health Care. All rights reserved.

UK164020611